European Representative:
Lotus Global Co., Ltd.
1 Four Seasons Terrace
West Drayton, Middlesex
London, UB7 9GG
United Kingdom
Tel: +0044-20-75868010
Fax: +0044-20-79006187

Made in China

Taian Dalu Medical
Instrument Co., Ltd.

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DE

- It is recommended that the spacer should be replaced with a new one after one year of use
- Please use according to physicians or licensed health care professionals instructions
- This device is designed to be used in home, hospital and clinic environments
- Open the package and carefully examine the product for damage, missing parts or foreign objects
- Remove any foreign objects prior to use
- If the patient's symptoms worsen please seek medical attention
- Do not leave the spacer unattended with children
- To be used with metered dose inhaler only
- This spacer is unsterilized and for single patient use only

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
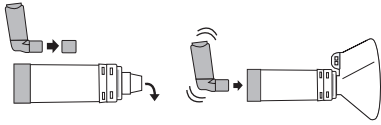
EN - Remove cap from the inhaler and shake the inhaler, then insert inhaler into the spacer as shown.

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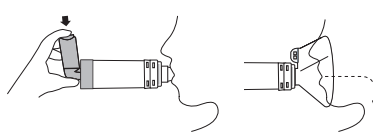
EN - Place lips around mouthpiece (if mask is not used), press inhaler; if using mask, assemble the mask spacer.

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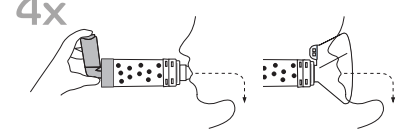EN - Breathe in and out through the mouthpiece 4 times, if more than one dose is required, repeat steps 2-3. If mask is used, place the mask over the mouth and nose, then press the inhaler; breathe in and out more through mask more than 4 times to make sure the medication inside the spacer is finished. If more than one dose is required, repeat steps 2-3.

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
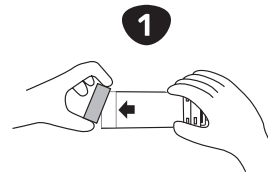
EN - Clean the spacer at least once a week by following steps (do not clean with boiling or hot water as this will damage your spacer):

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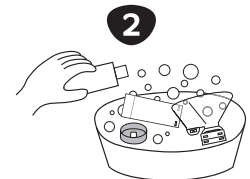EN - Remove the back piece and disassemble the other parts of the spacer.

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EN - Soak both parts for 15 minutes in lukewarm water with liquid detergent. Agitate gently.

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