OVULES INTIMES NON HORMONAUX

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COMPOSITION

-Composant actif : Hyaluronic Acid.

-Autres composants : Hydrogenated palm kernel glycerides, Hydrogenated Coco-Glycerides, Stearyl Heptanoate, Stearyl caprylate, Galactooligosaccharides, Tocopheryl acetate.

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NON-HORMONAL INTIMATE OVULES

Hydrates, heals and lubricates the vaginal mucosa

Box of 10 x 2.3g vaginal ovules, individually packaged.

Mucogyne® vaginal ovules containing hyaluronic acid are formulated to restore the vaginal mucosa in women of any age. The hyaluronic acid in Mucogyne® can maintain the natural suppleness, lubrication and moisturising of the vulvovaginal mucosa and compensate for insufficient physiological secretions. Clinical trials have demonstrated the capacity of hyaluronic acid to accelerate and improve the tissue repair process. Mucogyne® vaginal ovules have been specially formulated to respect the physiological pH of the intimate area.

WHEN TO USE MUCOGYNE® OVULES

For **HYDRATION** and a **LUBRICANT**, to deal with the problem of vaginal dryness, whatever the cause (with or without signs of vulva irritation, discomfort or itching): women who suffer pain during sexual intercourse, women taking the mini pill, mothers during breastfeeding, menopausal women, women taking anti-acne treatment.

-For **HEALING ADDITIVE**, for women with a damaged vaginal mucosa or who have undergone medical treatment (dystrophy following chemotherapy, burns following radiotherapy), after giving birth or after localised surgery.

WHAT IS THE DOSAGE OF MUCOGYNE® OVULES?

Mucogyne® ovules are inserted in the vagina:

- For **HYDRATION**, in the event of vulvo-vaginal dryness: 1 ovule twice a week for 5 weeks, preferably in the evening before bed, or as prescribed by your doctor.

- For **HEALING ADDITIVE**: 1 ovule per day for 10 days, to be repeated as needed.

HOW TO USE MUCOGYNE® OVULES

A. Remove the ovule from its packaging by pulling on the two tabs.

B. Insert the ovule deep into the vagina, preferably while lying down. The easiest way is to lie on your back, with your knees bent and legs apart.

HOW TO STORE MUCOGYNE® OVULES

Store medicines away from light and heat, preferably between 4°C and 25°C. Medicines should not be disposed down the drain or with household waste. Ask your pharmacist what to do with any unused medicines. These measures help to protect the environment.

PRACTICAL ADVICE:

We recommend you use a panty liner during treatment in case of discharge.

PRECAUTIONS FOR USE

For adult use only. Vaginal route. Single use. Do not swallow. Avoid contact with the eyes. Keep out of the reach of children. Wash hands well, before and after each use. Do not use in the event of allergy or hypersensitivity to one of the ingredients. Do not use the product after the expiry date or if it is damaged.

Ask a health care professional for advice before using Mucogyne® ovules during pregnancy, while menstruating, in the event of vaginal infection or if symptoms persist.

Mucogyne® is not a contraceptive method and may be used in conjunction with any hormonal treatment. Compatible with condoms.

UNDESIRABLE EFFECTS

Allergy to one of the ingredients.

COMPOSITION

- Active ingredient: Hyaluronic Acid.

- Other ingredients: Hydrogenated palm kernel glycerides, Hydrogenated Coco-Glycerides, Stearyl Heptanoate, Stearyl caprylate, Galactooligosaccharides, Tocopheryl acetate.

Mucogyne® is also available as a non-hormonal vaginal gel - hydrates, heals and lubricates

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